

Freedom 631™ LSO SPINAL ORTHOSIS


MODEL: F631-001

Indications:

Chronic Back Pain - Chronic Lumbar Instability - Herniated Lumbar Disc - Degenerative Disc Disease - Spondylosis Stenosis - Post-Operative Rehabilitation

APPLICATION AND CARE INSTRUCTION

1. Apply the Freedom 631™ in a standing position.
2. Open the LSO spinal orthosis and apply after centering the posterior panel and assuring that the bottom is resting on top of the buttocks.
3. Once in place, close the orthosis with the rigid anterior panel centered in the front.
4. Pull straps upward and outward at a 45° angle
5. Keeping the tension, secure the straps overtop of front

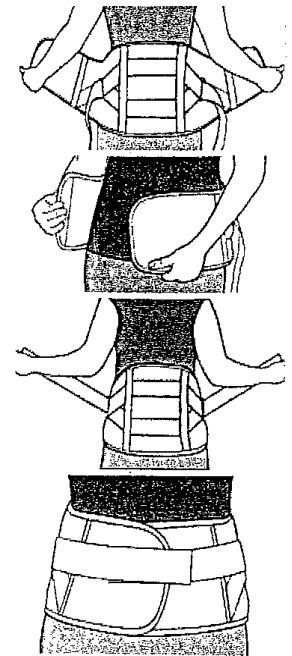

WEARING INSTRUCTIONS

Wear as prescribed by your physician. The Freedom 631™ can be worn under or over clothing.

CARE INSTRUCTIONS

Remove the plastic panel on the posterior of the orthosis and hand-wash the brace only as needed in cool water with a mild liquid soap. For posterior panel, remove the mesh cover and hand-wash the plastic with a mild topical cleaner. Make sure to lay flat and air dry completely before re-applying.

Consult a physician if pain, swelling, or unusual reactions are experienced. This device is only one element in the overall treatment program. There is no guarantee that injury will be cured or prevented through use of this product.