

LPR 2000™ LSO SPINAL ORTHOSIS


MODEL: C637-001

Indications:

Chronic Back Pain - Chronic Lumbar Instability - Herniated Lumbar Disc - Degenerative Disc Disease - Spondylosis Stenosis - Post-Operative Rehabilitation

APPLICATION AND CARE INSTRUCTION

1. Attach side panels onto the left and right side of anterior Velcro.
2. Apply the LPR2000™ in a standing position.
3. Open the LSO Spinal Orthosis and apply after centering the posterior panel and assuring that the bottom is resting on top of the buttocks.
4. Once in place, close the orthosis with the rigid anterior panel centered in the front.
5. Pull straps upward and outward at a 45° angle
6. Keeping the tension, secure the straps overtop of front.

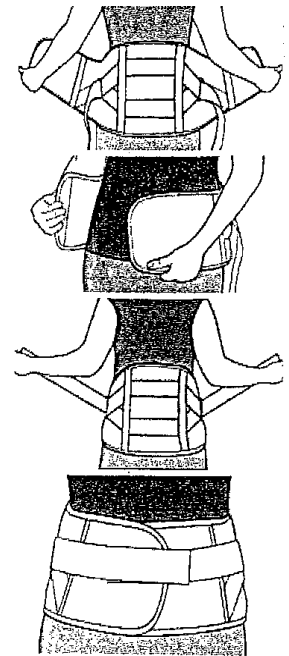

WEARING INSTRUCTIONS

Wear as prescribed by your physician. The LPR2000™ can be worn under or over clothing.

CARE INSTRUCTIONS

Remove plastic panel and hand wash the LPR2000™ as needed in warm water with a mild detergent. Pat dry to remove excess water. Air dry away from heat. Do not tumble dry. Do not bleach. Ensure completely dry before reapplying.

Consult a physician if pain, swelling, or unusual reactions are experienced. This device is only one element in the overall treatment program. There is no guarantee that injury will be cured or prevented through use of this product.